



## **Quick Reference to Calibrate Tracer**

1. Find the small black plastic brief case.
2. You'll find a metal bar with 2 cut outs with 162.83 marked at the top.
3. Place the bar into the tracer with 162.83 facing up and to the top then press the trace button.
4. Pull data onto the edger, on most edgers when you press the menu button once the circumference size will read at the top. If it's more than .05 off so 162.77 or 162.87 your sizing is off and you need to calibrate.
5. With the bar still in the tracer press and hold the L button on the tracer, within 5 seconds of pressing the L button press and hold the R button then let them both go. Now the center both eye button light will be flashing. Press the center button and the tracer will start to calibrate.
6. Repeat step # 4 and confirm the tracer is on size.
7. Get in the routine of doing this every Monday just to keep the sizing true.

Any questions please contact [theFinishingLab](mailto:Info@theFinishingLab.com) at [Info@theFinishingLab.com](mailto:Info@theFinishingLab.com)  
or call us Direct at 510.815.4466